



DRMS AirMaxx™ CAM Walker

AirMaxx CAM Walker (Low Top)

AirMaxx CAM Walker (High Top)



Features

- **Rocker bottom to simulate natural gait**
- **Large air bladder creates a larger surface for contact pressure to help control swelling and ensure a snug fit.**
- **Product available in other styles, materials, and configurations, including with or without pneumatic pump.**

Instructions

To Put on DRMS AirMaxx CAM Walker:

1. Open the boot liner without removing it from the boot.
2. Place the boot flat on the floor and put the foot inside the boot. Make sure the heel is all the way to the back of the boot.
3. Secure the lining to the leg by overlapping the liner on the top of the foot and at the front of the lower leg.
4. Secure the Velcro straps beginning with the straps closest to the toes.

To Control the Air Pressure:

1. To put air inside the boot, turn the valve clockwise until secure, and then squeeze the bulb until you feel adequate pressure.
2. To remove air from the boot, simply turn the valve counterclockwise.

AirMaxx CAM Walker Boot (High Top)	AirMaxx CAM Walker Boot (Low Top)	Size	Men's Shoe Size	Women's Shoe Size
AB0004-141-00	AB0004-128-00	X-Small	< 4.0	< 5.5
AB0004-141-01	AB0004-128-01	Small	4.5 - 7.0	6.0 - 8.0
AB0004-141-02	AB0004-128-02	Medium	7.5 - 10.0	8.5 - 11.5
AB0004-141-03	AB0004-128-03	Large	10.5 - 12.5	11.5 - 13.5
AB0004-141-04	AB0004-128-04	X-Large	12.5 <	13.5 <